

OUSON Health

Everyday health records,
in one place

iPhone and iPad

Healthy Life, Happy Lives



A clear view of everyday readings


Five measurements

Blood pressure, temperature, blood oxygen, blood glucose and weight.


One overview

Latest readings, recorded times and a quick view of weight changes.

iPhone

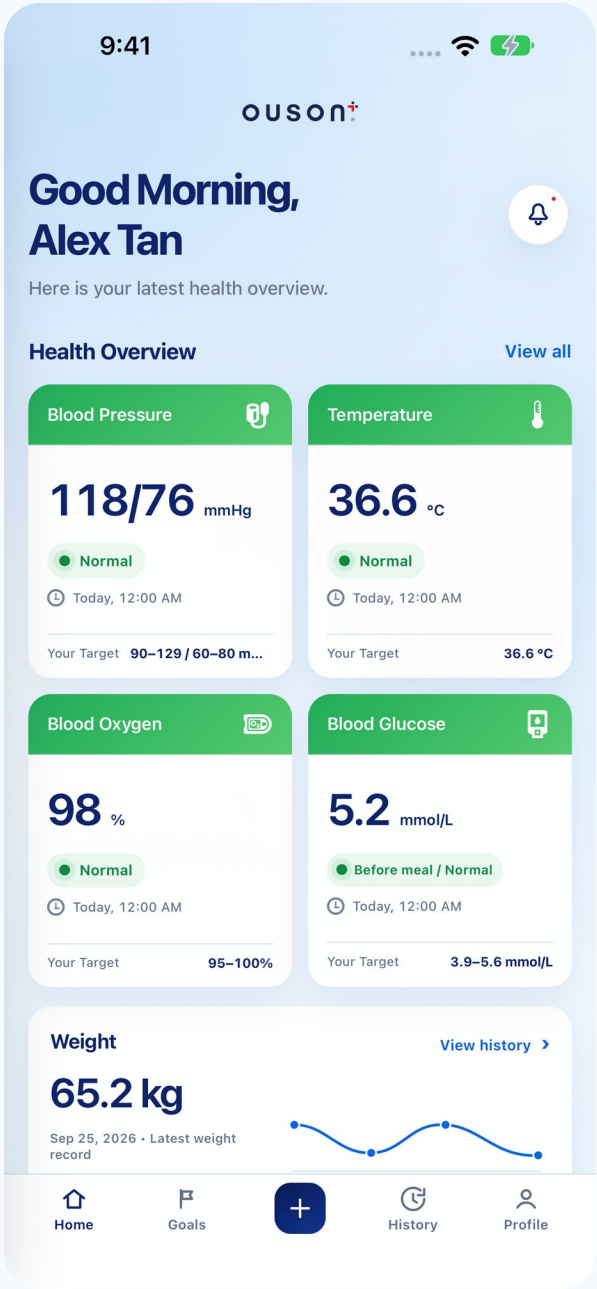

iPad

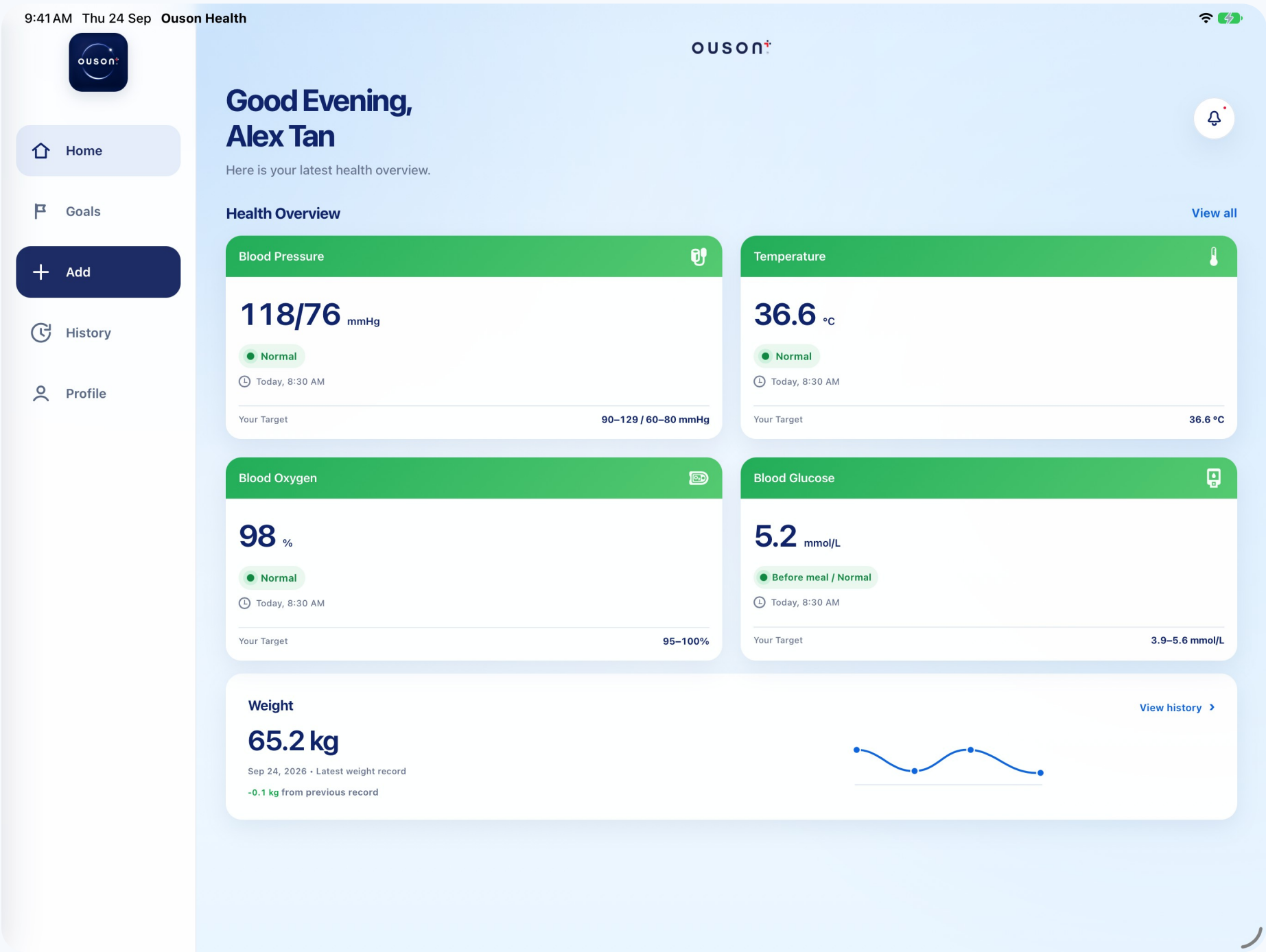

SIGN IN AND ACCOUNT SETUP


A familiar way to get started

Flexible sign-in
Email, Google or Apple.


Account setup
Create an account and complete email verification.

Recovery when needed
Forgot Password helps users regain access.

iPhone

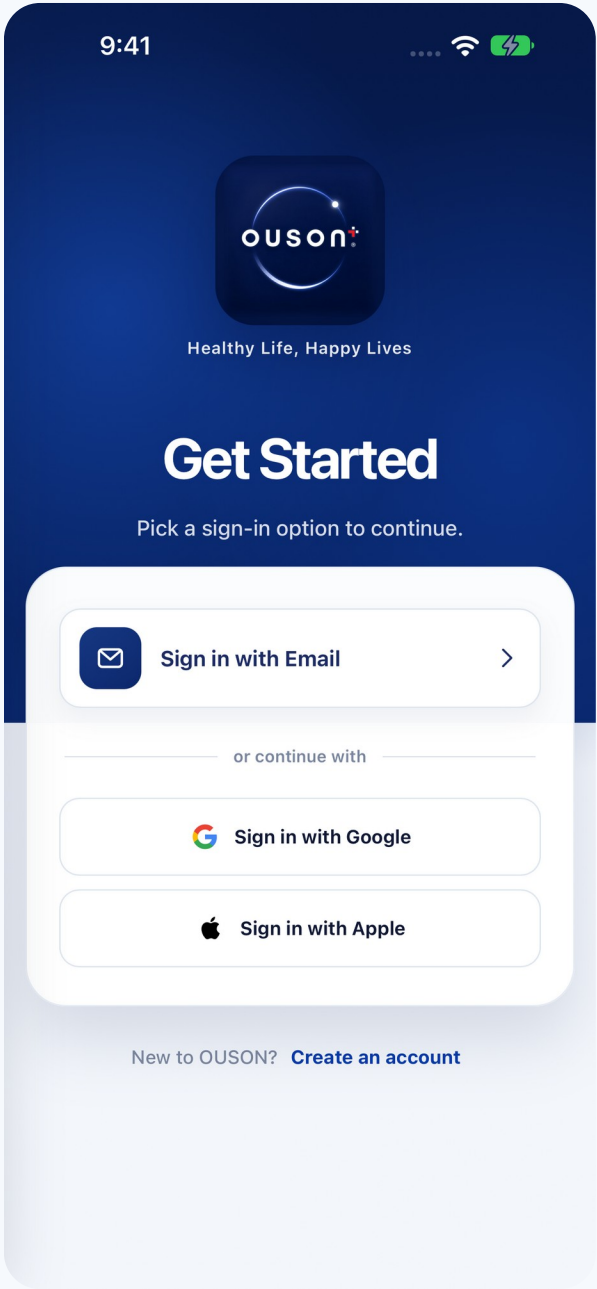

iPad

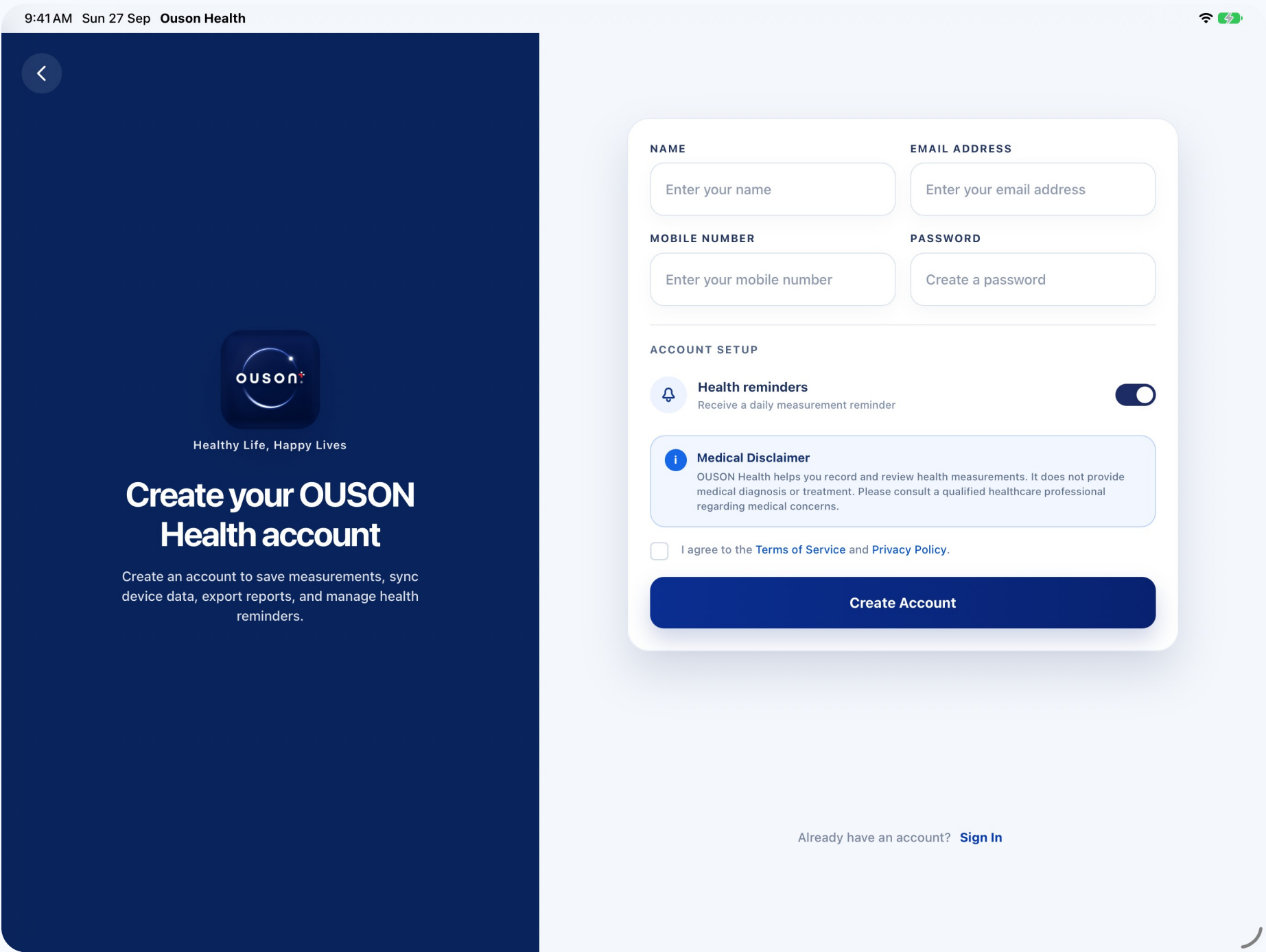

ADD MEASUREMENT

Less typing with Smart Scan

Manual entry

Select a measurement, add its value, date, time and an optional note.

Camera-assisted entry


Copy displayed readings from supported OUSON BSX516 and BSX556 monitors.

Review before saving


Compare the result with the monitor and correct it if needed.

Smart Scan reads the monitor display. It does not measure blood pressure.

iPhone

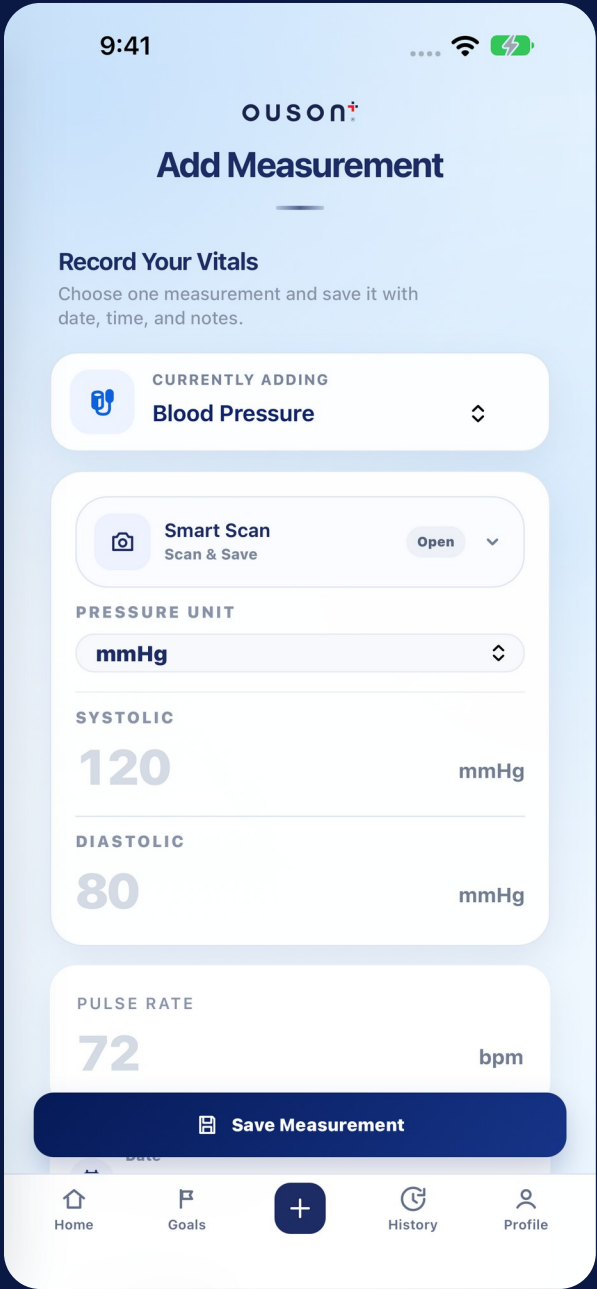

iPad

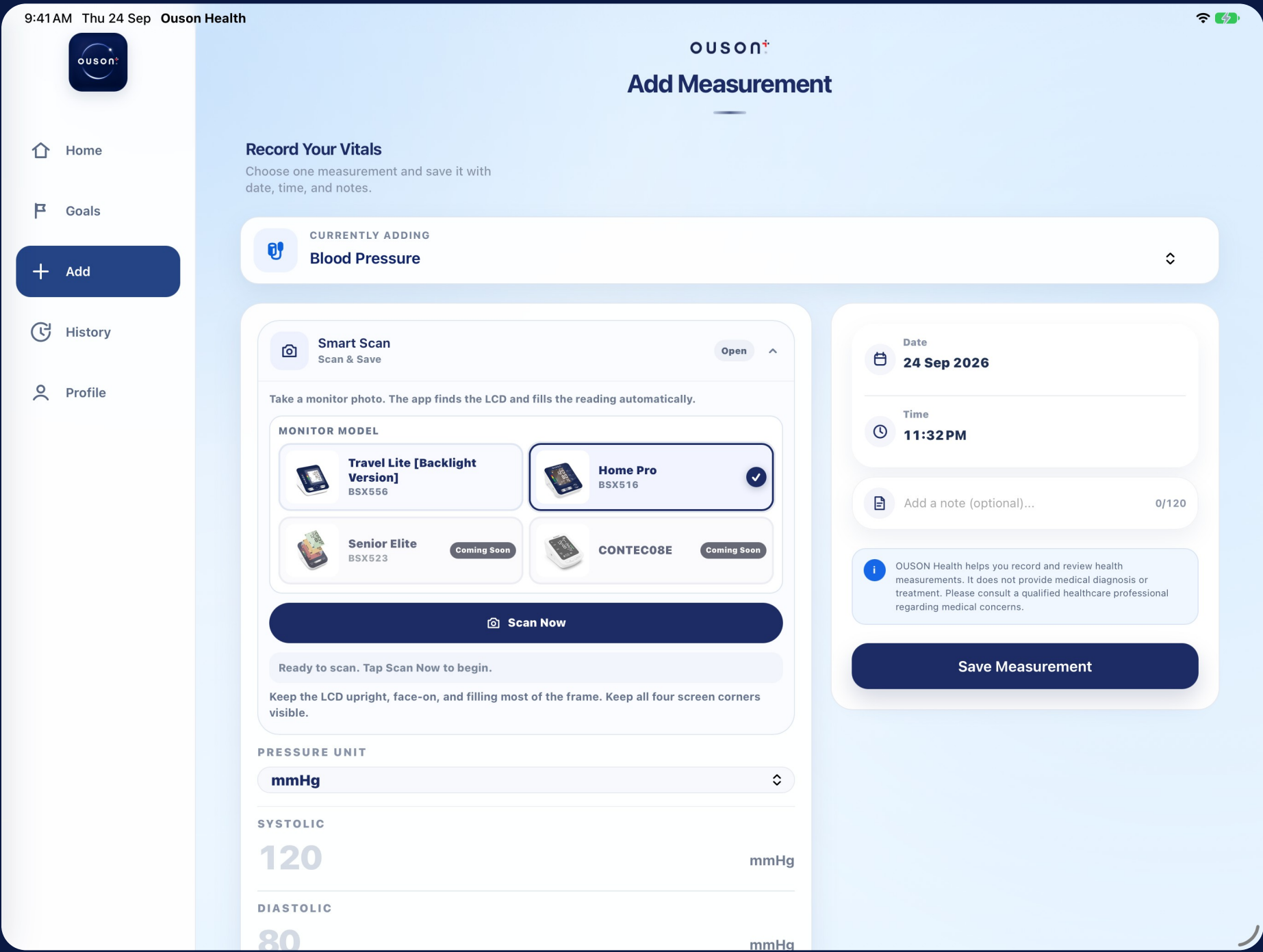

HEALTH GOALS

Personal ranges, visible progress

Personal targets

Set supported target ranges to suit the user’s tracking needs.

Progress at a glance


Compare the latest reading or a seven-day view with the saved range.

Temperature uses a general reference. Medical targets should follow professional advice.

iPhone



iPad

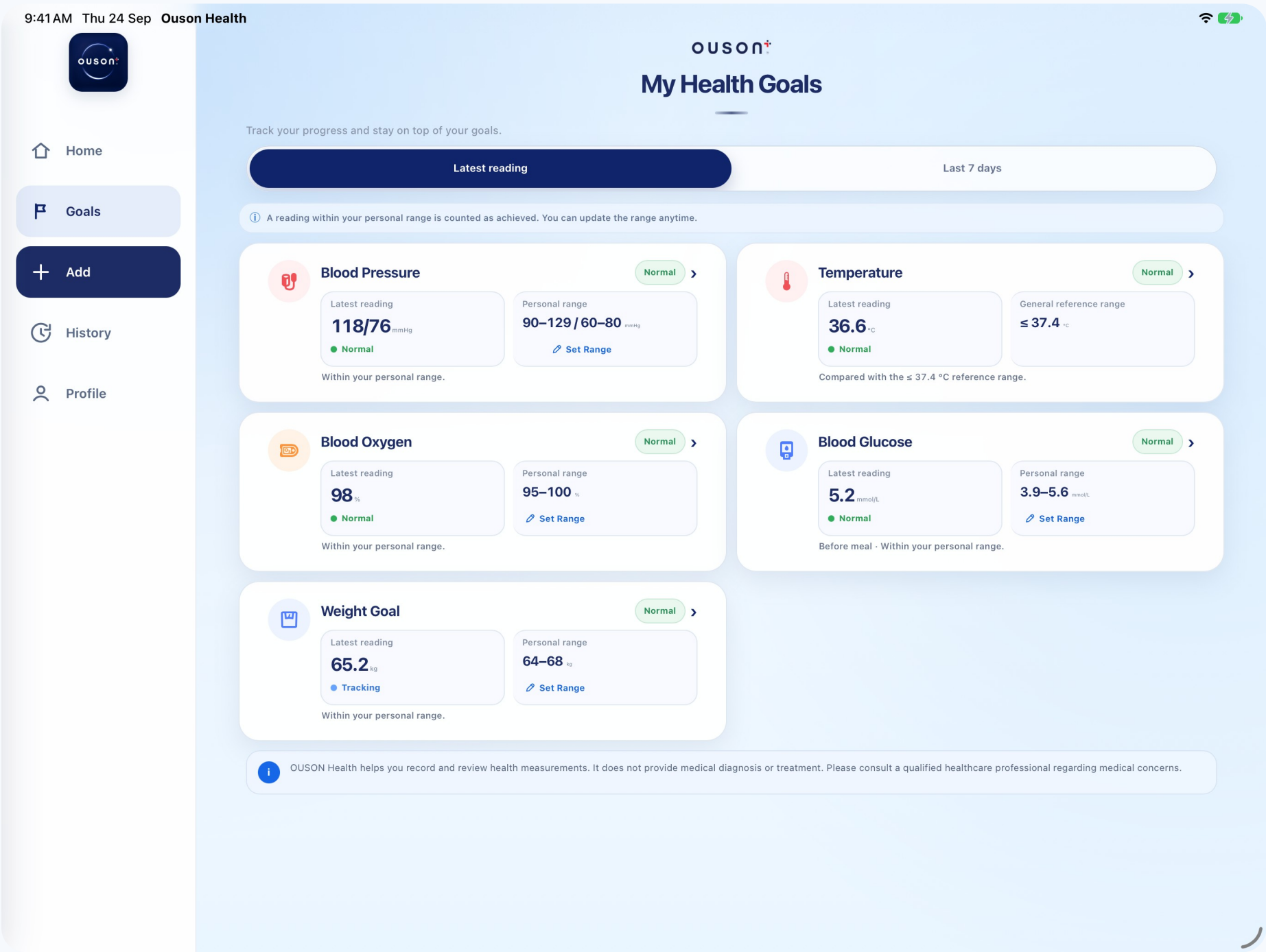

HISTORY

The story behind each reading

Trends over time

Switch measurements and review seven-day or thirty-day trends.

Details stay available

See saved readings, filter by month and remove an incorrect record.


Blood pressure summary

View the latest reading and the latest three-reading average.

iPhone



iPad

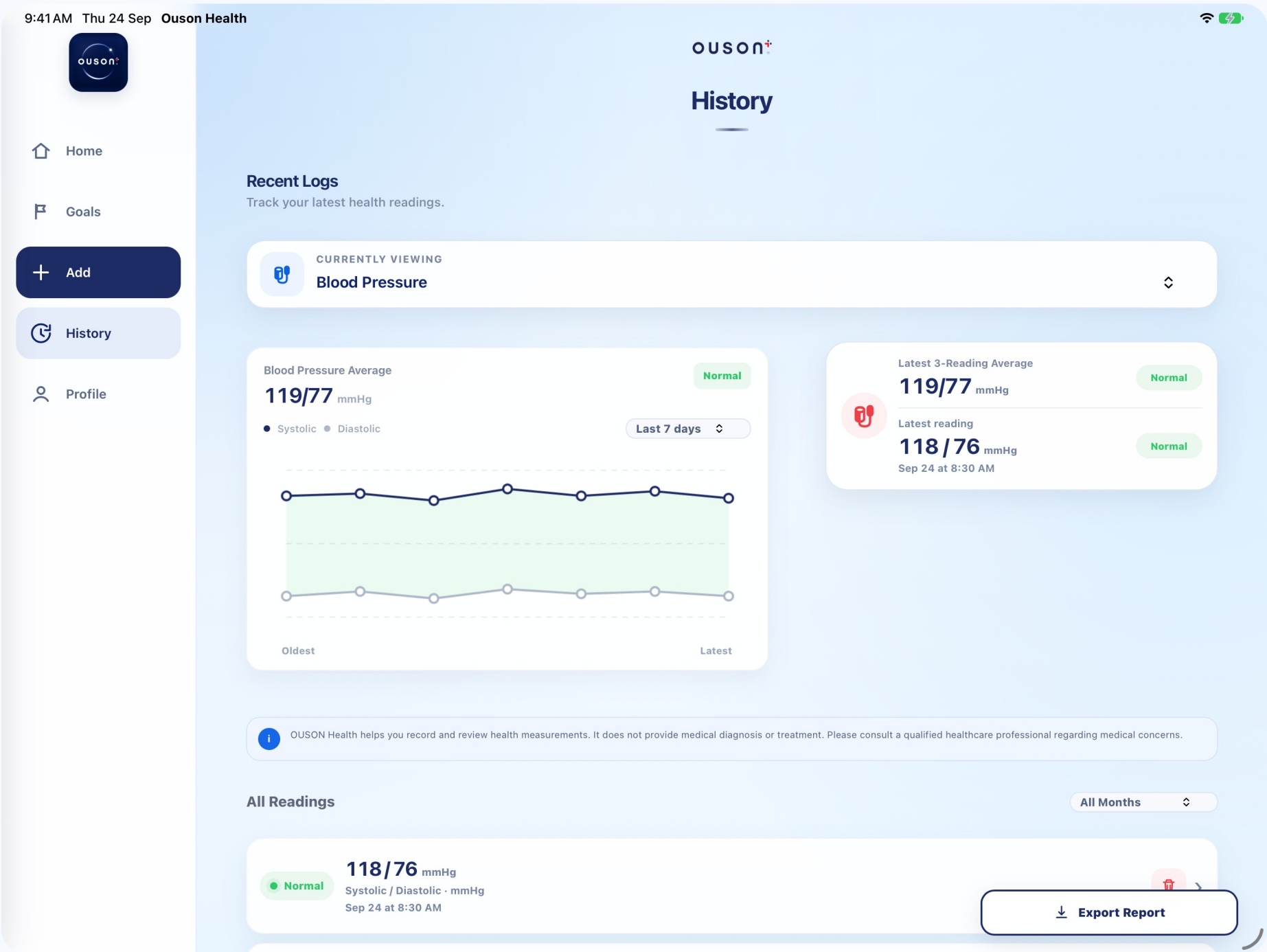


EXPORT HEALTH REPORT

Records that are easy to share


Choose the period
Last seven days, last thirty days or a custom date range.

A PDF for discussion
Export the selected measurement for a conversation with a healthcare professional.

iPhone

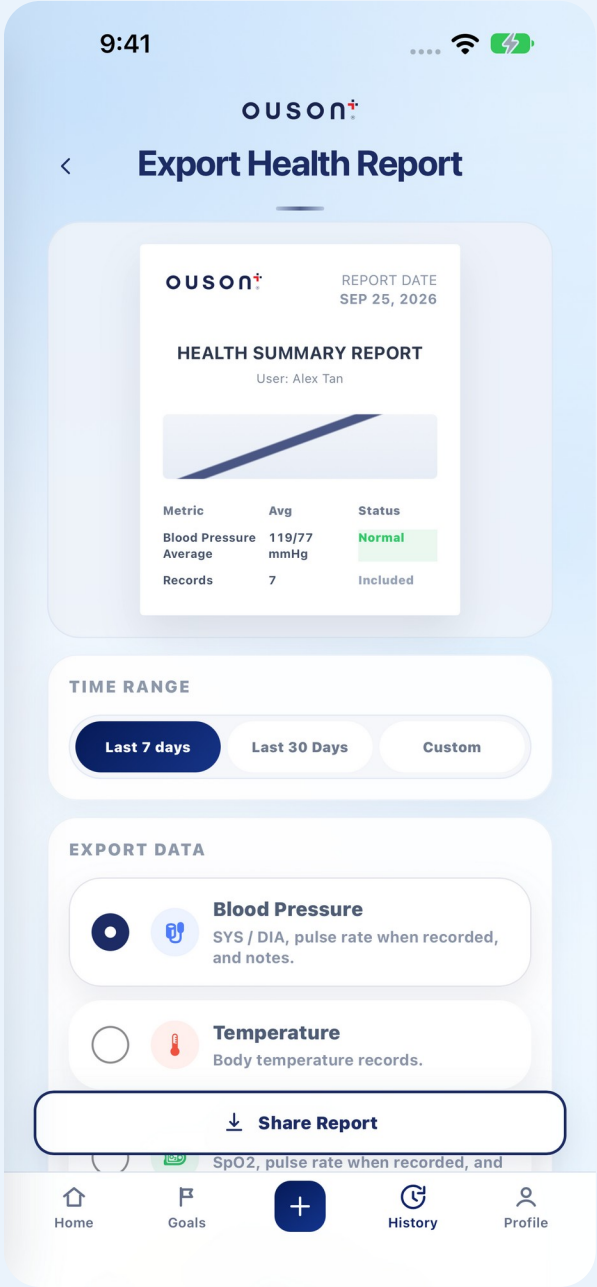

iPad

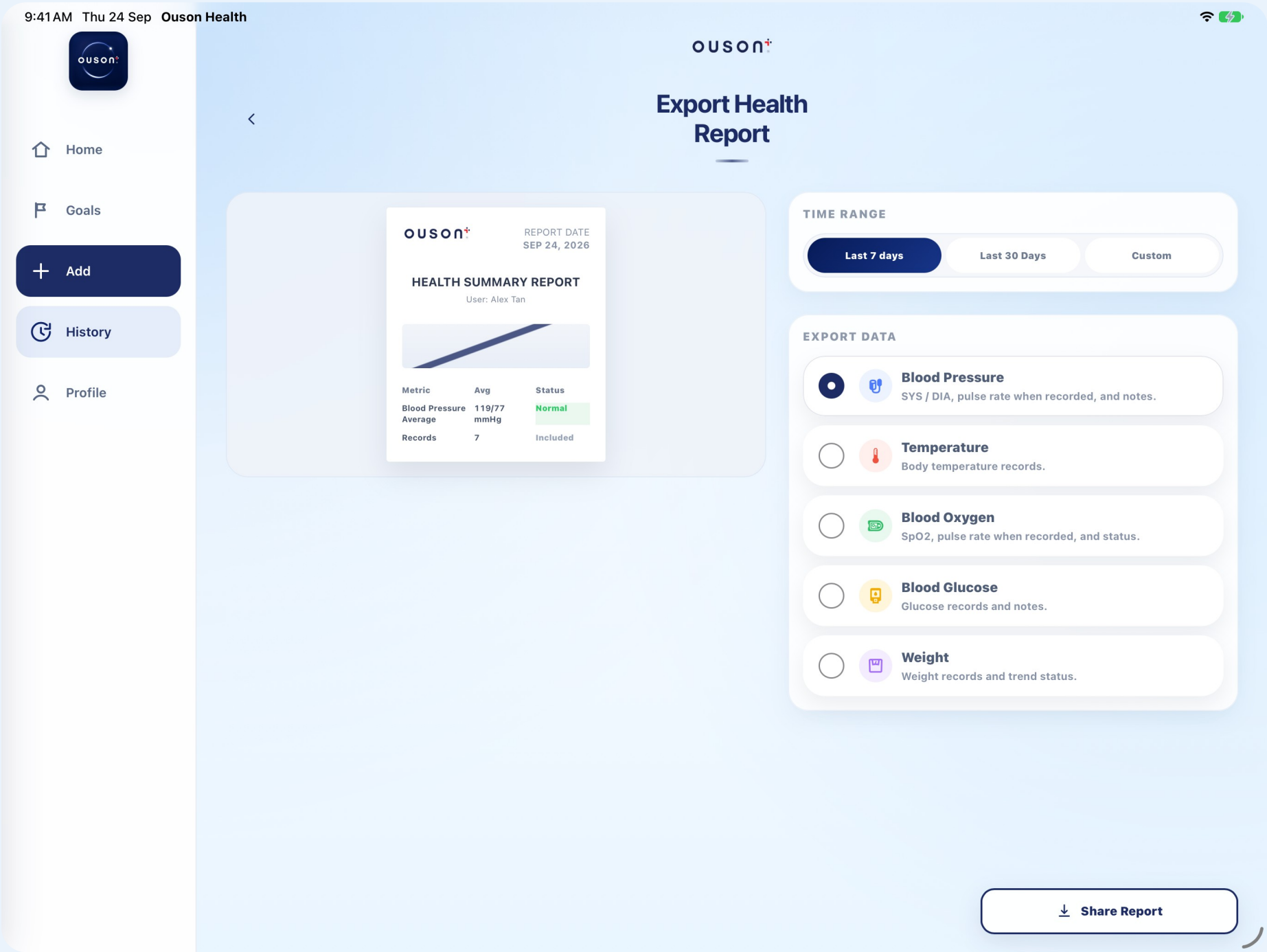

NOTIFICATIONS

A routine that fits the day

Separate schedules


Choose reminder times for each measurement type.

More than one time


Add or remove daily reminder times as routines change.

Reminders depend on notification permission and device settings.

iPhone

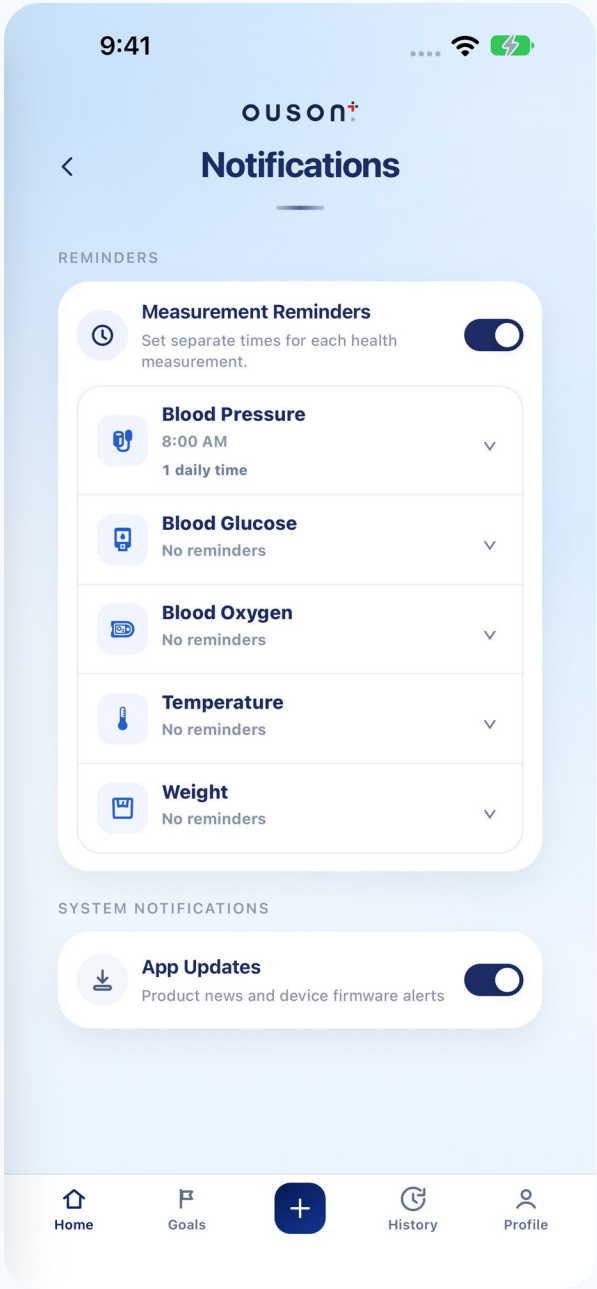

iPad

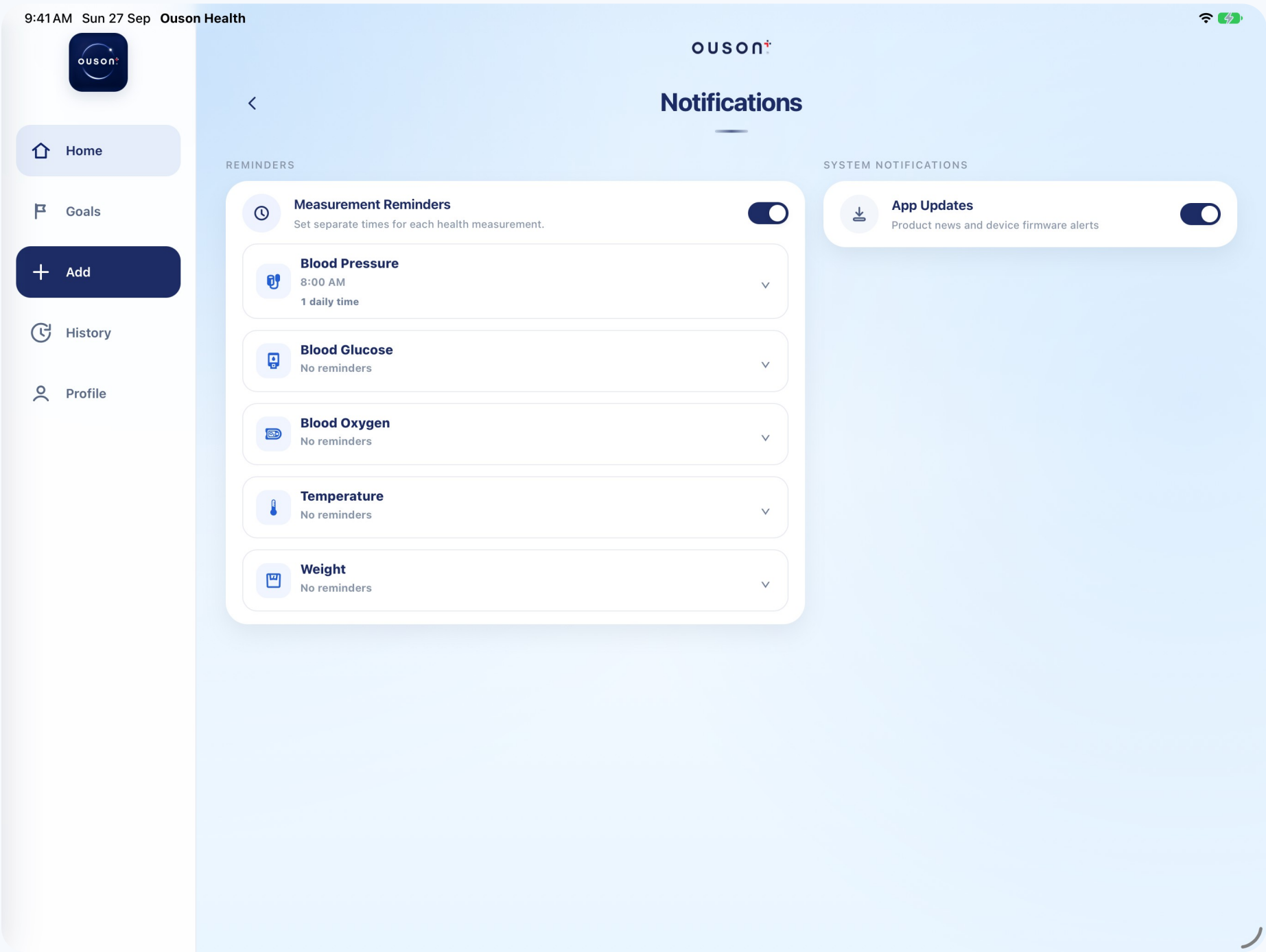

MY DEVICES

Device details close at hand

A device overview


Keep registered OUSON products and warranty details together.

Registration support


Add product and purchase details, with model, serial and lot guidance.

Warranty status and eligibility follow the applicable registration terms.

iPhone

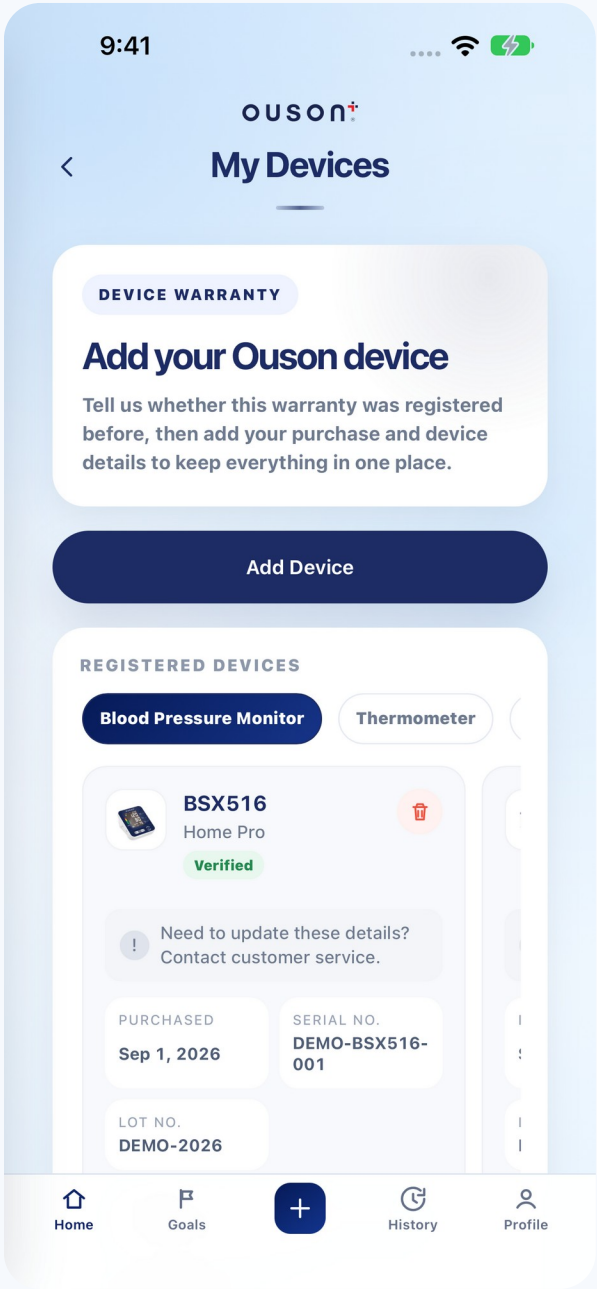

iPad

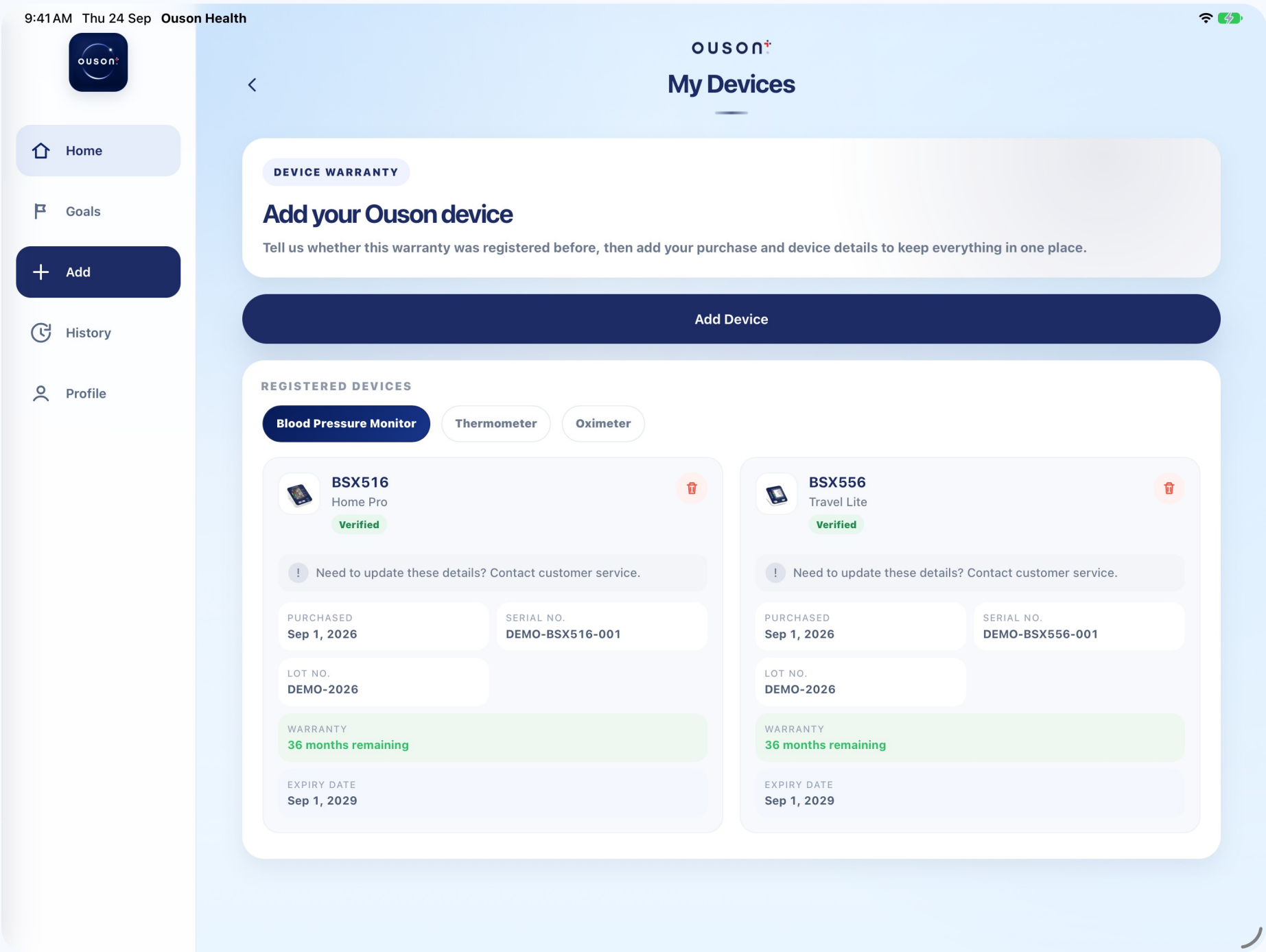

PROFILE AND SETTINGS

An app that feels personal

Personal details

Update profile information and photo.


Three languages

English, Bahasa Melayu and Chinese.


Account controls

Change password, request account deletion and read the Privacy Policy.

iPhone

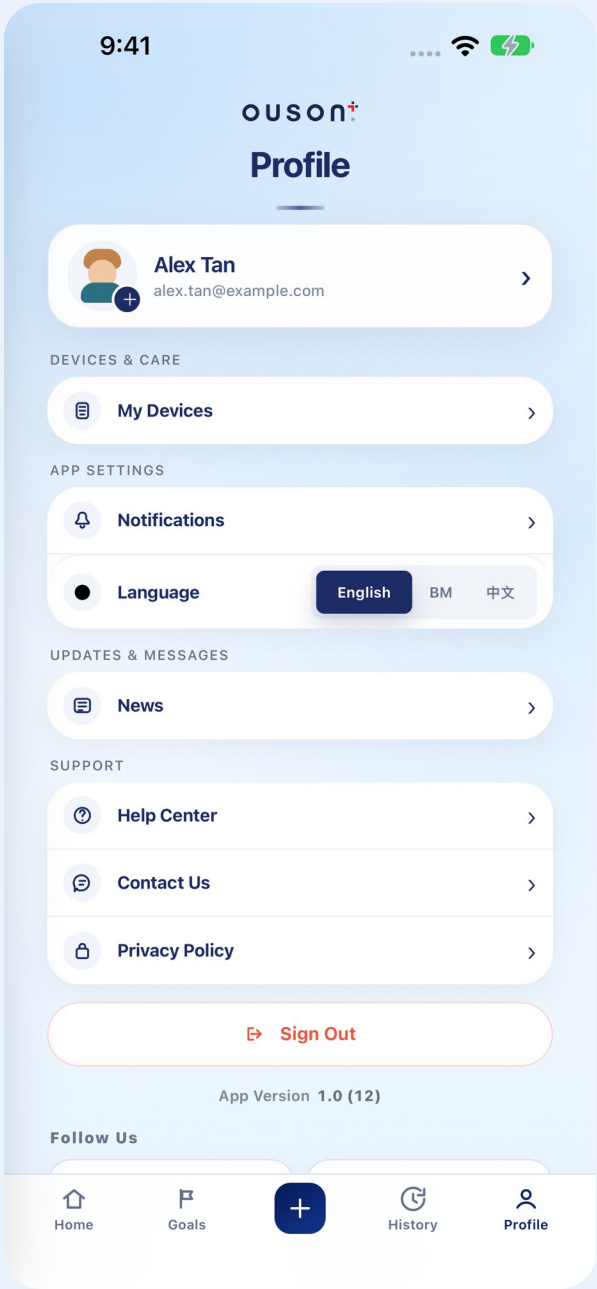

iPad

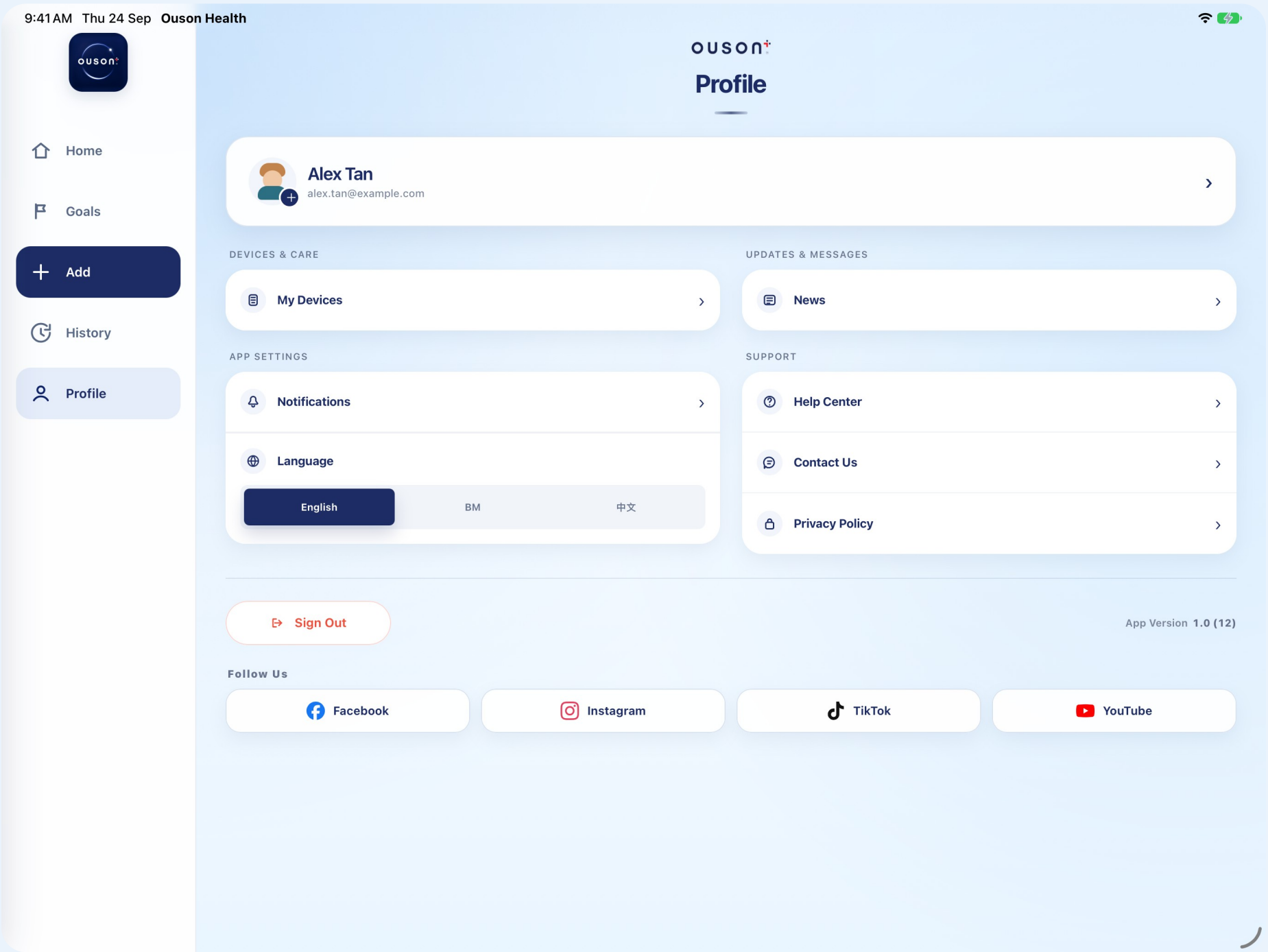

Help throughout the product journey

Product guidance

User guides, illustrated steps, manuals and troubleshooting.


App guidance

Feature explanations and answers to common questions.


A route to the team

WhatsApp support and OUSON news in the app.

iPhone

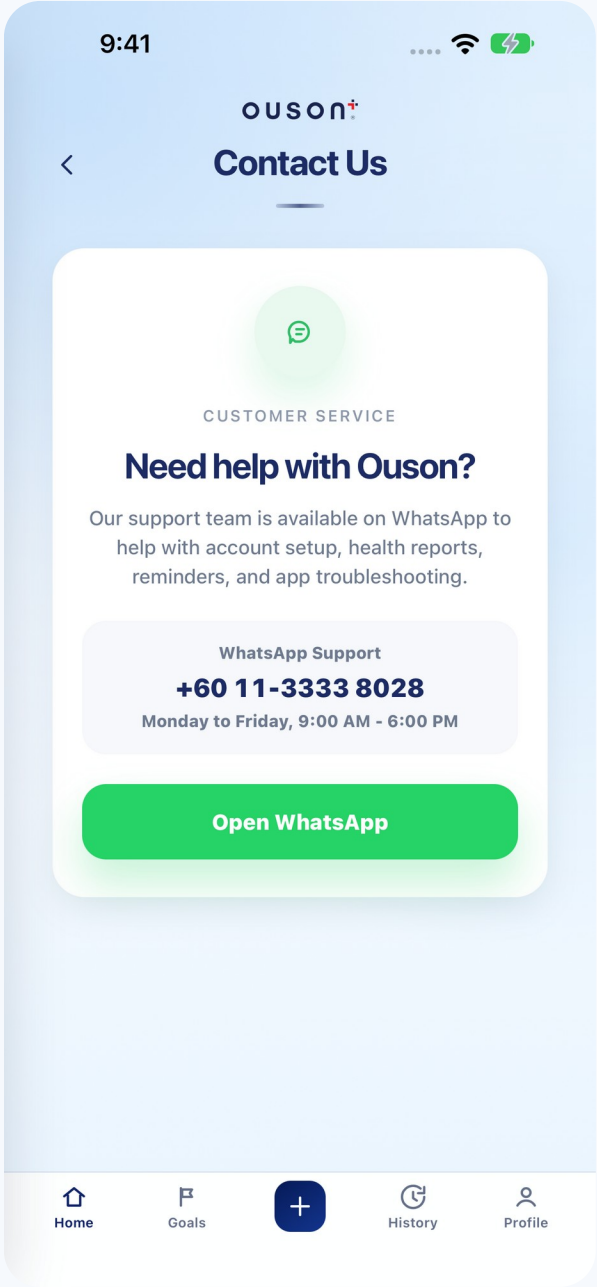

iPad

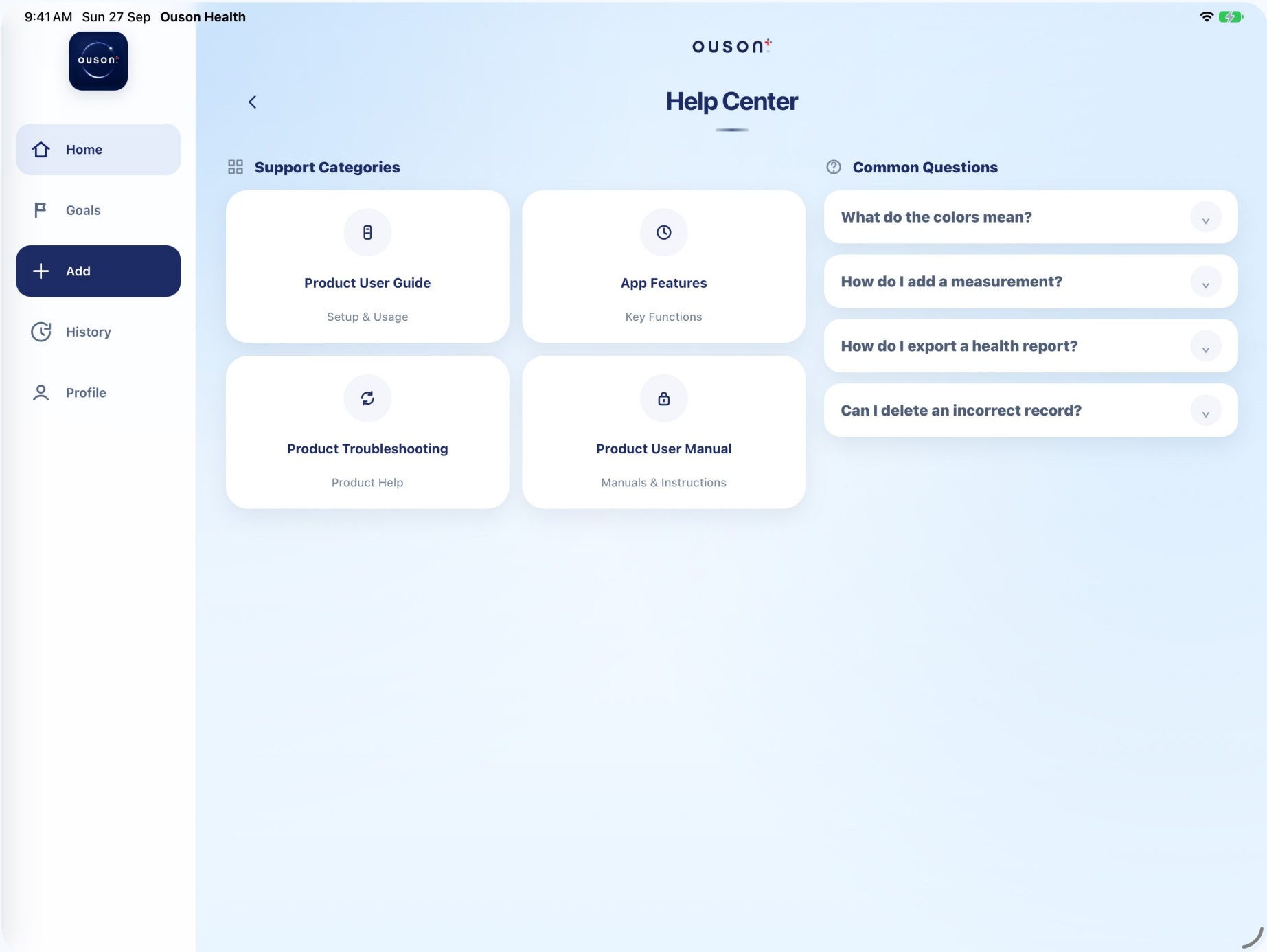



OUSON Health

Healthy Life, Happy Lives

Personal health records on iPhone and iPad

www.ouson.com.my

For recording and reviewing measurements. No medical diagnosis or treatment.
Consult a qualified healthcare professional before medical decisions.